

Cautions

- Refer to your therapy device manual for details on settings and operational information.
- The technical specifications of the mask are provided for your clinician to check that they are compatible with the therapy device. If used outside specification, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak or variation in the leak rate may affect device function.
- The mask should be cleaned every other day to prevent dirt and oil contamination from the skin building up on the mask.
- The mask should not be cleaned with unapproved cleaning or disinfecting methods.
- The user should clean their face prior to wearing the mask.
- The user should not alter the design of the mask.
- Do not place the mask in a clothes dryer.
- In the USA, federal law restricts this device to sale by, or on the order of, a physician.

Contraindications

- None

Technical Specifications

1. Therapy Pressure Range: 4 to 20cmH₂O
2. 22mm swivel connection
3. Resistance (pressure drop)
@ 50 lpm = 0.6 cmH₂O
@ 100 lpm = 0.9 cmH₂O
4. Fixed Leak rate (flow may vary due to manufacturing variation):

Pressure (cmH ₂ O)	4	5	10	15	20
Flow (lpm)	12.9	14.7	24.9	33.9	40.0

5. Environmental storage and operating conditions are not expected to effect the function of this device
6. Sound Level
 - A-weighted Sound Power Level, 26 dBA
 - A-weighted Sound Pressure Level @ 1m, 18 dBA
7. This device is expected to remain suitable for use under normal usage and cleaning conditions for a time period of at least 90 days.
8. The device is not made with natural rubber latex

Limited Warranty www.circadiance.com/help



EMERGO EUROPE
Prinsessegracht 20,
2514 AP The Hague,
The Netherlands



0459



SleepWeaver
Soft Cloth Masks

Advance

Instructions for Use


English

Intended Use

This SleepWeaver mask is intended to provide an interface for Continuous Positive Airway Pressure (CPAP) or bi-level therapy. This mask is intended for single-patient reuse in the home and multi-patient, multi-use in the hospital/institutional environment. This mask is to be used on patients greater than 66 lbs (30 kg).



Fitting the Mask

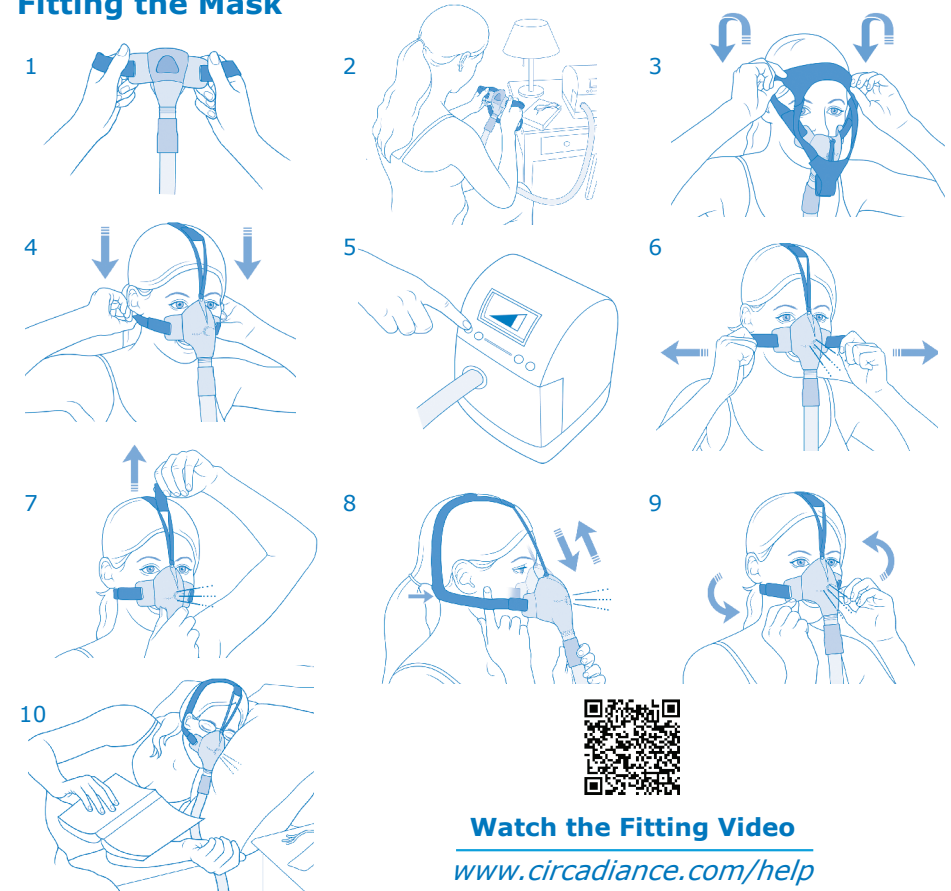

Watch the Fitting Video
www.circadiance.com/help



Address: 1010 Corporate Lane Export, PA 15632
Phone: 888-825-9640 or +1 724-858-2837
info@circadiance.com www.sleepweaver.com

