


ANEW FULL-FACE MASK FITTING GUIDE:

Facial Measurements for Full-Face Masks

(ruler or tape measure needed – see fig. 1)

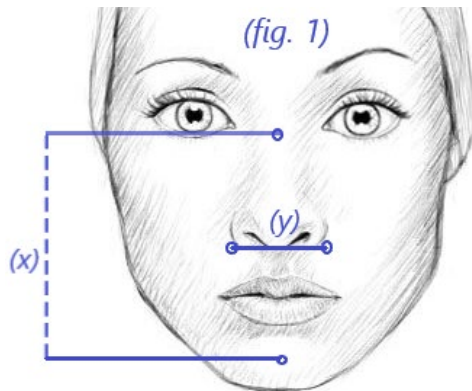

To determine your SleepWeaver full face mask size, measure the width at the bottom of your nose, edge to edge. Enter the width (y) in inches (round to closest quarter): _____ in.

Next, measure from the bridge of your nose (usually just below your eye line) to the center (i.e., dimple) of your chin. Enter the length (x) in inches (round to closest quarter): _____ in.

NOTE: It may be helpful to have a friend assist you. For accurate results of the face, do not use a flexible measuring device such as a sewing measuring tape or a string.



Anew For full face masks, we recommend taking two measurements of your face and using the *smaller* of the two for sizing.

1) First measure underneath your nostrils. If you measure 1 inch, try the **Small Anew**. If you measure 1½ inches, try the **Regular Anew**. If you measure 1½ inches or more, try the **Large Anew**.

2) The second measurement is to measure vertically from the bridge of your nose to the middle of your chin. If you measure 3¼ to 3½ inches, try the **Small Anew**. If you measure 3½ to 3¾ inches, try the **Regular Anew**. If you measure 3¾ inches or more, try the **Large Anew**.